careBrush

EN Instructions for self-collection of vaginal specimens under the guidance of a medical professional

Intended Use

For use with the careHPV® Test. The careBrush is intended for use with careHPV Collection Medium for the collection of specimens for subsequent testing with the careHPV Test. Vaginal specimens may be self collected under the guidance of a medical professional, and cervical specimens may be collected by physicians or trained medical professionals. Important: The instructions in this document are for the self-collection of vaginal specimens under the guidance of a medical professional. For instructions on the collection of cervical specimens by a medical professional, refer to the document provided with each package of careBrushes.

Read these instructions for use (IFU) carefully before collecting a vaginal specimen.

Refer to careHPV Test Kit Handbook for additional information on specimen handling.

Please visit support.qiagen.com for your local QIAGEN Representative's contact information. Use this contact information to attain technical support or to report a serious incident.

Materials Provided

- 1 careBrush
- 1 IFU

Materials Required But Not Provided

1 careHPV Collection Medium tube

Warnings and Precautions

- Do not use the careBrush on pregnant women.
- Do not use if package is damage
- Do not re-use.
- Do not collect a specimen if anti-fungal cream or high concentrations of blood, contraceptive jelly or douche are present.
- Do not use after the expiration date indicated next to the  symbol on the package label.
- Specimens to be tested using the careHPV Test must be collected using a careBrush and stored in a careHPV Collection Medium tube contain ing careHPV Collection Medium.
- Specimens collected with a careBrush and stored in careHPV Collection Medium cannot be used for culture.
- Do not use collected specimens for testing if the careHPV Collection Medium spills before or after the vaginal sample is obtained. If a spill occurs, collect a new specimen and place the careBrush into a new careHPV Collection Medium vial.
- Consider all human specimens potentially infectious.
- Dispose of all specimens, reagents and other potentially contaminated materials in accordance with national and local egulations.
- Do not touch the brush before or after specimen collection.
- Do not touch the brush to any surface outside of the collection tube.

- Do not put the careHPV Collection Medium in your mouth or ingest; if the liquid gets in your mouth or is ingested, immediately notify a medical professional.
- Do not inhale the careHPV Collection Medium; if inhaled, immediately notify a medical professional and get fresh air.
- Do not allow the careHPV Collection Medium to contact your eyes; if the liquid contacts your eyes, immediately notify a medical professional and keep your eye open while rinsing for several minutes under running water.
- Self-collection is intended for use under the guidance of a medical professional. Consult with your doctor or nurse if you need any advice or information on the device, if you have difficulties using the device or if you think that the device may have led to an undesirable side-effect not listed in these instructions for use.

Storage and Handling

Store unopened careBrushes at 15–30°C. Product stored as directed will retain the expiration date printed on the package label.

Procedure

Important: If at any time during this procedure you experience pain or major discomfort, stop and consult your medical professional.

- Taking care not to spill the liquid in the tube, remove the cap from the careHPV Collection Medium tube and place the cap on a hard, flat surface with the inside of the cap pointing up (1).
- Gently open the outside of your vagina and insert the brush into the opening (2).
- Continue to insert the brush by gently rotating the brush to the left or to the right until you feel resistance (3).
- Hold the brush in place and gently rotate the brush in a circle 5 times (4).
- Remove the brush from your vagina and put the brush in to the careHPV Collection Medium tube so the brush touches the bottom of the tube and is immersed in the liquid.
- Hold the tube firmly with one hand and use your other hand to grip the brush shaft. Break off the end of the shaft by snapping the shaft at the line (5). Only the lower portion, with the brush side down, should be in the liquid.
- Discard the broken portion of the brush shaft. Leave the remainder of the careBrush in the careHPV Collection Medium.
- Place the cap on the careHPV Collection Medium tube. Ensure the cap is pushed flush and the careHPV Collection Medium tube is tightly sealed (6).
- Clearly label the tube with your first and last name and the collection date.

Specimen Handling and Storage

Give the specimen to a medical professional as soon as possible. Specimens collected with a careBrush and placed in careHPV Collection Medium tubes may be shipped and stored at room temperature (15–30°C) for 14 days or at 2–8°C for 35 days. For domestic transport, package and label specimens in

compliance with applicable national and local regulations covering the transport of clinical specimens and infectious substances.

Glossary of Symbols

	Temperature Limitation		Manufacturer
	CE - IVD marked Symbol		Do not reuse
	Do not use if package is damaged		Sufficient for "X" preparations
	Consult instructions for use		Use by
	Batch code		Open here
	Catalog number		Global Trade ID Number

For a full Glossary of Symbols, see the careBrush Instructions For Use in the careBrush kit box.

Ordering Information

Product	Contents	Cat. no.
careBrush	50 pre-scored cervical brushes	619024
careHPV Collection Medium	50 tubes containing 1 ml of careHPV Collection Medium	619025
careHPV Test For 96 reactions; kit for detection of 14 high-risk HPV genotypes		614015

Trademarks: QIAGEN®, Sample to Insight®, careHPV® (QIAGEN Group). CARE is a registered trademark of COOPERATIVE FOR ASSISTANCE AND RELIEF EVERYWHERE, INC. ("CARE"). CARE and the members and affiliates of CARE International are not affiliated with QIAGEN and do not sponsor, endorse, support, participate in or control the development, manufacture, use or sale of any QIAGEN product. Registered names, trademarks, etc., used in this document, even when not specifically marked as such, are not to be considered unprotected by law. © 2020 QIAGEN, all rights reserved.

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QIAGEN GmbH