



AmniSure® ROM Test

AmniSure ROM Test Quick Reference Guide

AmniSure ROM Test quick facts (1)

- AmniSure is a rapid, non-instrumented, qualitative immunoassay to aid in the detection of ROM (Rupture Of [fetal] Membranes)
- 98.9% sensitive and 98.1% specific
- Not affected by urine, semen, vaginal infections and trace amounts of blood on the swab
- No speculum exam required
- No gestational age limit
- Billable with CPT® 84112

Proper sample collection tips (1)

- Collect specimen from patients presenting with signs, symptoms or complaints suggestive of ROM
- Specimen should be tested within 4 hours of collection*
- Collect specimen prior to digital examination or lubricants
- Collect specimen prior to use of any disinfectant solutions or medicines or 6 hours after their removal

The AmniSure 4-step test procedure*

1

Collect sample


1 minute collection.
Collect sample of vaginal discharge with sterile collection swab (no speculum required).



2

Transfer to solvent

1 minute dilution.
Rinse specimen swab in solvent vial.
Discard swab.

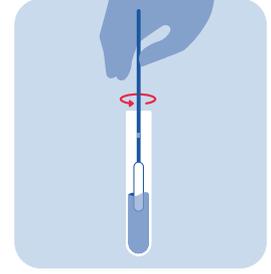

3

Insert test strip

Insert test strip into vial to initiate PAMG-1 detection process.
After 10 minutes, remove the test strip, and read the results within 5 minutes.




4

Read results


Remove test strip from vial, observe and record results. Do not read strip after 15 minutes have passed since dipping strip into vial.

Note: Faint or broken lines should always be read as positive.

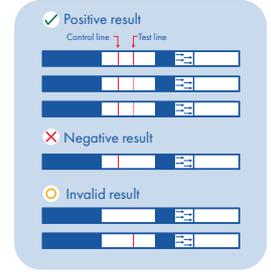


✓ Positive result: Two lines


Control line Test line

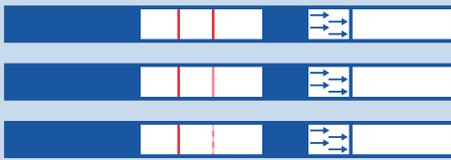

✗ Negative result: One control line

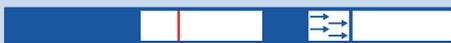

○ Invalid result: No lines or test line only

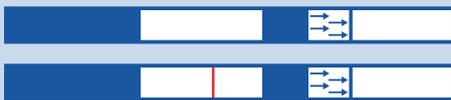

Additional notes for reading the test results:

- Faint or broken lines should always be read as positive.



- The presence of no lines or only a test line indicates an invalid test result. DO NOT interpret this as a negative test result. Invalid results require a retest.

Important limitations of use (1):

- Failure to detect membrane rupture does not assure the absence of membrane rupture.
- Results should be used in conjunction with other clinical information.

References:

- AmniSure ROM Test Instructions for Use, QIAGEN, 1090607, Rev. 06, 02/2022. Available at www.qiagen.com.

For up-to-date licensing information and product-specific disclaimers, see the respective QIAGEN® kit instructions for use or user manual. QIAGEN kit instructions for use and user manuals are available at www.qiagen.com or can be requested from QIAGEN Technical Services or your local distributor.

*If the patient sample is not tested within 4 hours and sample storage is necessary, tightly close the sample vial and place in a refrigerator. Do not test the sample after more than 6 hours have passed since sample collection.



Learn more at www.qiagen.com or contact an AmniSure specialist at AmniSureNA@qiagen.com.

Trademarks: QIAGEN®, Sample to Insight®, AmniSure® (QIAGEN Group), CPT® (American Medical Association). Registered names, trademarks, etc. used in this document, even when not specifically marked as such, are not to be considered unprotected by law. PROM-10956-005 1121020 05/2023 © 2023 QIAGEN, all rights reserved.

Ordering www.qiagen.com/shop | Technical Support www.support.qiagen.com | Website www.qiagen.com