

Important Note

Recommended upgrade to QIAstat-Dx® Gastrointestinal Panel 2 (IVDR) ADF version 1.1

Dear laboratory partner,

Your satisfaction with the QIAstat-Dx system is our highest priority. That's why we're releasing an upgraded Assay Definition File (ADF) for **QIAstat-Dx Gastrointestinal Panel 2** (cat. no. 691413). **Please note that this recommended update will only take 1 minute to complete.**

This recommended update to ADF version 1.1 (V1.1) will provide significant improvements to enhance your experience in working with the QIAstat-Dx system.

To use the new ADF, you need to import the QIAstat-Dx Gastrointestinal Panel 2 ADF version 1.1 onto your QIAstat-Dx Analyzer 1.0, QIAstat-Dx Analyzer 2.0 or QIAstat-Dx Rise. The QIAstat-Dx Gastrointestinal Panel 2 ADF V1.1 is compatible with QIAstat-Dx Analyzer 1.0 running software version 1.4 or 1.5, with QIAstat-Dx Analyzer 2.0 running software version 1.6 or later and with QIAstat-Dx Rise running software version 2.4 or later.

How to update your QIAstat-Dx Gastrointestinal Panel 2 ADF to version 1.1 and import to your QIAstat-Dx Analyzer System

For QIAstat-Dx Analyzer 1.0 and QIAstat-Dx Analyzer 2.0:

The following steps for update and import of ADF can be performed by customers. If you experience any issues during the update, please contact QIAGEN Technical Services at www.support.qiagen.com.

1. Request the ADF file from your QIAGEN sales representative or go to www.qiagen.com/QIAstat-Dx-ADF.
2. In the **Resources** tab, click **Protocol Files** and download the **QIAstat-Dx Gastrointestinal Panel 2 ADF version 1.1** file. Save and unzip the package. Copy the ***.asy** file to the root folder of the USB storage device (directly on the USB storage device, not in any folder).

3. Insert the USB storage device that contains the corresponding ADF into the USB port of the QIAstat-Dx Analyzer 1.0 or the QIAstat-Dx Analyzer 2.0.
4. Press **Options > Assay Management**. The Assay Management screen appears in the content area of the display.
5. Press the **Import** icon located at the bottom left of the screen.
6. Select the ADF file to be imported from the USB storage device. Then, confirm the upload of the file in the dialog box that appears.
7. If a dialog box appears asking you to overwrite the current version with the new one, press **Yes** to confirm.
8. Select **Assay Active** to enable the assay to become active.
9. Perform the following steps to assign the active assay to a user:
 - a. Go to **Options > User Management**.
 - b. Select the user who should be allowed to run the assay.
 - c. From the User Options section, select **Assign Assays**.
 - d. Enable the assay, then press **Save**.

For QIAstat-Dx Rise: Please contact Technical service or your sales representative to upload new assay definition files.

For additional information, refer to the Instructions for Use of the specific assay (QIAstat-Dx Gastrointestinal Panel 2). If you have any further questions regarding the abovementioned points, please do not hesitate to contact us via your QIAGEN sales representative or your local QIAGEN Technical Services at www.support.qiagen.com. We are here to support you.

With kind regards,

QIAGEN

www.qiagen.com