digene® Female Swab Specimen Collection Kit

en i READ ALL INSTRUCTIONS FULLY AND CAREFULLY BEFORE USING THIS PRODUCT.

The CE marking indicates that the digene® Female Swab Specimen Collection Kit meets the requirements of European Directive 93/42/EEC relating to medical devices.

Specimen Collection

Note: Collect Pap smear specimen before obtaining specimen for DNA testing. Collect DNA specimen prior to application of acetic acid or iodine if a colposcopy will be performed.

1. Remove excess mucus from the cervical os and surrounding ectocervix using 1 of the 2 specimen collection swabs provided. Discard the swab in accordance with local regulations.
2. Insert the swab into the endocervical canal and rotate it through 5–180° turns, alternating in opposite directions (1).
3. Rub swab firmly over the entire area of the transformation zone (2).
4. Withdraw the swab carefully; avoid contact with vaginal mucosa.
5. Insert the swab to the bottom of the transport tube. Snap off the shaft at the score line (3), and cap the tube securely (4).

⚠ Please read additional storage and handling instructions on reverse.

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1 x 1 ml [STM]

[RX ONLY]

[en] For collection and transport of cervical specimens to be tested only with digene® Hybrid Capture 2 CT/GC DNA Tests.

Storage Instructions
Store kits at 15-30°C. Do not use after the expiration date indicated next to the symbol on the pouch label.

Some kit components contain sodium azide, which may react with lead or copper plumbing to form highly explosive metal azide compounds. Dispose of in accordance with local regulations. Contact your local QIAGEN Representative to obtain Material and Safety Data Sheets.

Specimen Transport Medium: Warning! Causes mild skin irritation. If skin irritation occurs: Get medical advice/ attention.

Specimen Transport
Cervical samples may be held for up to 2 weeks at room temperature and shipped without refrigeration. Store specimens at 2-8°C upon receipt at the laboratory and test within 1 week; otherwise, store specimens at -20°C for up to 3 months prior to testing.

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Trademarks: QIAGEN®, digene®, Hybrid Capture® (QIAGEN) ©2015 QIAGEN all rights reserved.
[EC REP] QIAGEN GmbH QIAGEN Strasse 1 40724 Hilden GERMANY