

September 2015

cattletype[®] BHV1 gB Ab Handbook



5 (catalog no. 270043)



20 (catalog no. 270045)

For detection of antibodies to
glycoprotein B of bovine herpesvirus 1

Registered in accordance with § 17c of the German
Law on Animal Diseases (FluB 491)

REF

270043, 270045



QIAGEN Leipzig GmbH, Deutscher Platz 5b,
04103 Leipzig, Germany

Contents

Kit Contents	3
Intended Use	4
Symbols	5
Storage	5
Safety Information	6
Quality Control	6
Introduction	7
Principle	7
Equipment and Reagents to Be Supplied by User	8
Important Notes	9
General precautions	9
Protocol: ELISA Test Procedure	10
Test procedure for serum and plasma samples	11
Test procedure for milk samples	13
Data Interpretation	14
Troubleshooting Guide.....	16
Appendix: Quick Guide.....	17
Ordering Information.....	18

Kit Contents

<i>cattle</i> type BHV1 gB Ab	(5)	(20)
Catalog no.	270043	270045
Number of plates	5	20
Test Plate: microtiter plate with 96 wells, coated with inactivated BHV1 antigen	5	20
Sample Diluent, ready to use	1 x 30 ml	1 x 125 ml
Negative Control, ready to use	1 x 3.5 ml	2 x 3.5 ml
Positive Control, ready to use	1 x 3.5 ml	2 x 3.5 ml
Wash Buffer (10x)	3 x 125 ml	2 x 500 ml
Conjugate, ready to use	1 x 60 ml	1 x 240 ml
TMB Substrate, ready to use	1 x 60 ml	1 x 240 ml
Stop Solution, ready to use	1 x 60 ml	1 x 240 ml
Handbook	1	1

Intended Use

The *cattletype* BHV1 gB Ab is an enzyme immunoassay (ELISA). It is intended for the detection of antibodies to glycoprotein B of bovine herpesvirus 1 (BHV1) in serum, plasma, and milk samples from cattle. It can be used with individual samples or with pooled milk samples after sample preparation using the *cattletype* Milk Prep Kit. The kit is approved by the Friedrich-Loeffler-Institut and registered in accordance with § 17c of the German Law on Animal Diseases (FLI-B 491) for use in Germany for veterinary diagnostic procedures.

For veterinary use only.

Symbols

 Σ	<N>	Contains reagents for <N> plates
		Legal manufacturer
	LOT	Lot number
		Use by date
		Temperature limitations for storage
	i	Handbook
	REF	Catalog number
	MAT	Material number
		Protect from light
		For cattle samples

Storage

The components of the *cattletype* BHV1 gB Ab ELISA should be stored at 2–8°C and are stable until the expiration date stated on the label. Wash Buffer (10x) and Stop Solution may be stored at

room temperature (18–25°C) to avoid salt crystallization. If test strips are provided with the kit, store the remaining test strips in the re-sealed foil pouch with desiccant at 2–8°C until next use. The test strips can be stored for at least 6 weeks after opening the plate pouch.

Safety Information

When working with chemicals, always wear a suitable lab coat, disposable gloves and protective goggles. For more information, please consult the appropriate safety data sheets (SDSs). These are available online in convenient and compact PDF format at www.qiagen.com/safety where you can find, view and print the SDS for each QIAGEN kit and kit component.



CAUTION: The Stop Solution contains 0.5 M sulfuric acid.

All sample residues and objects that have come into contact with samples must be decontaminated or disposed of as potentially infectious material.

Quality Control

In accordance with QIAGEN's ISO-certified Quality Management System, each lot of *cattletype* BHV1 gB Ab is tested against predetermined specifications to ensure consistent product quality.

Introduction

The *cattletype* BHV1 gB Ab is a highly sensitive solution for the detection of antibodies to glycoprotein B of bovine herpesvirus 1 (BHV1). BHV1 is the causative agent of Infectious Bovine Rhinotracheitis (IBR) – a respiratory disease with tracheitis, rhinitis, and fever. In addition BHV1 infections can cause Infectious Pustular Vulvovaginitis (IPV), balanoposthitis, and abortions. Clinical disease is often followed by latent BHV1 infection. Reactivation of the virus can be the cause of spreading of the infection in the herd. The *cattletype* BHV1 gB Ab permits the detection of anti-BHV1 antibodies in serum, plasma, and milk samples of cattle infected with BHV1 or vaccinated with a glycoprotein B containing vaccine.

Principle

cattletype BHV1 gB Ab is a blocking ELISA. The Test Plate is coated with inactivated BHV1 antigen. During sample incubation, BHV1-specific antibodies bind to the immobilized antigen. Unbound material is removed by rinsing. The HRP-labeled, gB-specific monoclonal antibody is added, which cannot bind to the BHV1 antigen while its antigenic determinant is previously occupied by antibodies in the test sample. Unbound anti-gB-HRP conjugate is rinsed out. A color reaction is initiated by adding the substrate solution and stopped after 10 minutes. The optical density (OD) is measured in a spectrophotometer. The blocking value (percentage of inhibition) is calculated from the OD values obtained with the test sample and the Negative Control, which contains no BHV1-specific antibodies.

Equipment and Reagents to Be Supplied by User

When working with chemicals, always wear a suitable lab coat, disposable gloves and protective goggles. For more information, consult the appropriate safety data sheets (SDSs), available from the product supplier.

- Beakers
- Measuring cylinders
- Pipets (adjustable)
- Multichannel pipets (adjustable)
- Aluminum or adhesive foil for covering the Test Plate
- Optional: Device for delivery and aspiration of Wash Buffer
- Microtiter plate absorbance reader
- Tubes or plates for diluting the samples
- Distilled water

Important Notes

General precautions

The user should always pay attention to the following:

- Do not expose the TMB Substrate Solution to intense light or to sunlight when performing the test.
- Components of the test kit should not be contaminated or mixed with components from other batches.
- Do not use the components of the test kit past the expiration date.
- Water from ion-exchange systems used for diluting the Wash Buffer (10x) may interfere with the assay if not pure enough. Use double-distilled water or highly purified water (e.g., Milli-Q®).
- For accurate test results, it is essential to use clean glassware and to pipet and rinse carefully and strictly adhere to the incubation times when performing the test.

Protocol: ELISA Test Procedure

Important points before starting

- Please read “Important Notes” on page 9 before starting.

Things to do before starting

- Bring reagents to room temperature (18–25°C) immediately before use. In case of precipitated salt crystals in the Wash Buffer (10x), dissolve by gentle swirling and warming.
- Dilute Wash Buffer (10x) 1:10 in distilled water. For example, for one Test Plate dilute 50 ml Wash Buffer (10x) in 450 ml distilled water and mix.
- Fresh, refrigerated or previously frozen serum or plasma samples may be used.

Preparation of milk samples

Prior to sample analysis, milk samples have to be defatted. Centrifuge whole milk samples for 10 min at 3000 x g at 10°C or store samples at 2–8°C overnight. Then remove the cream.

Note: After sample preparation using a suitable concentration protocol, bulk milk samples can be tested with *cattletype* BHV1 gB Ab. The assay is validated for testing pools of up to 50 milk samples after bulk milk sample preparation using the *cattletype* Milk Prep Kit.

Test procedure for serum and plasma samples

Please read “Things to do before starting”, page 10.

Procedure

1. Pipet 50 μ l of ready-to-use Sample Diluent into the Test Plate wells.
2. Pipet 50 μ l Negative Control (in duplicate) and Positive Control (in duplicate) into appropriate wells and mix.
3. Pipet 50 μ l of the samples into remaining wells and mix.
Note: Record the positions of the controls and samples in a test protocol. Mix by either using a plate shaker or by repeated liquid aspirating and dispensing. Cover the Test Plate.
4. Incubate for 2 hours at 37°C or overnight (12–18h) at room temperature (18–25°C).
5. Remove solution from the wells by aspiration or tapping.
6. Rinse each well 5x with 300 μ l of prepared Wash Buffer. Remove the buffer after each rinse by aspiration or tapping.
7. Pipet 100 μ l ready-to-use Conjugate to each well and incubate for 60 min at room temperature (18–25°C).
8. Remove solution from wells by aspiration or tapping.
9. Rinse each well 5x with 300 μ l of prepared Wash Buffer. Remove the buffer after each rinse by aspiration or tapping.
10. Pipet 100 μ l TMB Substrate Solution to each well.
11. Incubate for 10 min at room temperature in the dark. Begin timing after the first well is filled.

-
12. Stop the reaction by adding 100 μ l Stop Solution per well. Add the Stop Solution in the same order as the Substrate Solution was added.
 13. Measure the OD in the plate reader at 450 nm within 20 min after stopping the reaction. Measuring at a reference wavelength (620–650 nm) is optional.

Test procedure for milk samples

Please read “Things to do before starting”, page 10.

Procedure

1. Pipet 100 μ l Negative Control (in duplicate) and Positive Control (in duplicate) into appropriate wells.
2. Pipet 100 μ l of defatted milk samples into remaining wells.
Note: Record the positions of the controls and samples in a test protocol. Cover the Test Plate.
3. Incubate overnight (12–18 h) at room temperature (18–25°C).
4. Remove solution from the wells by aspiration or tapping.
5. Rinse each well 5x with 300 μ l of prepared Wash Buffer. Remove the buffer after each rinse by aspiration or tapping.
6. Pipet 100 μ l ready-to-use Conjugate to each well and incubate for 60 min at room temperature (18–25°C).
7. Remove solution from wells by aspiration or tapping.
8. Rinse each well 5x with 300 μ l of prepared Wash Buffer. Remove the buffer after each rinse by aspiration or tapping.
9. Pipet 100 μ l TMB Substrate Solution to each well.
10. Incubate for 10 min at room temperature in the dark. Begin timing after the first well is filled.
11. Stop the reaction by adding 100 μ l Stop Solution per well. Add the Stop Solution in the same order as the Substrate Solution was added.
12. Measure the OD in the plate reader at 450 nm within 20 min after stopping the reaction.
Measuring at a reference wavelength (620–650 nm) is optional.

Data Interpretation

Validation criteria

The results are valid if the following criteria are met:

- The mean value (MV) of the measured OD value for the Negative Control (NC) must be ≥ 0.60 .
- The blocking value calculated from the MV of the measured OD value for the Positive Control (PC) must be $\geq 75\%$

In case of invalid assays, the test should be repeated after carefully reading the instructions for use.

Calculation

Calculate the MV of the measured OD for the Negative Control (NC) and the Positive Control (PC).

The blocking value is calculated according to the following equation:

$$\% \text{ blocking} = \frac{\text{MV OD}_{\text{NC}} - \text{OD}_{\text{sample}}}{\text{MV OD}_{\text{NC}}} \times 100$$

Interpretation of the results

- Samples with blocking values $<45\%$ are negative.
Specific antibodies to BHV1 could not be detected.
- Samples with blocking values $\geq 45\%$ and $<55\%$ are suspect.
It is recommended to retest animals with suspect results.
- Samples with blocking values $\geq 55\%$ are positive.
Specific antibodies to BHV1 were detected.

Troubleshooting Guide

The scientists in QIAGEN Technical Services are always happy to answer any questions you may have about either the information and/or protocols in this handbook or sample and assay technologies (for contact information, visit www.qiagen.com).

Appendix: Quick Guide

Sample dilution:

Serum/plasma 1:2, mix well

Milk Undiluted

Step	Short protocol	Overnight protocol*
1. Sample		100 µl/well
2. Incubation	Serum: 2 h 37°C	Overnight at RT
3. Wash		5 x 300 µl
4. Conjugate		100 µl/well
5. Incubation	60 min at room temperature	
6. Wash		5 x 300 µl
7. TMB		100 µl/well
8. Incubation	10 min at room temperature	
9. Stop		100 µl/well
10. Read		450 nm

* If using milk samples, incubation **must** be carried out overnight.

Data interpretation

Negative	Suspect	Positive
<45%	≥45% and <55%	≥55%

Ordering Information

Product	Contents	Cat. no.
<i>cattletype</i> BHV1 gB Ab (5)	For 480 reactions: 5 Test Plate (strips), Wash Buffer, Sample Diluent, Positive Control, Negative Control, Conjugate, TMB Substrate Solution, Stop Solution	270043
<i>cattletype</i> BHV1 gB Ab (20)	For 1920 reactions: 20 Test Plates , Wash Buffer, Sample Diluent, Positive Control, Negative Control, Conjugate, TMB Substrate Solution, Stop Solution	270045
Related products		
<i>cattletype</i> BHV1 gE Ab (5)	For 480 reactions: 5 Test Plates (strips), Wash Buffer, Sample Diluent, Positive Control, Negative Control, Conjugate, TMB Substrate Solution, Stop Solution	270203
<i>cattletype</i> BHV1 gE Ab (20)	For 1920 reactions: 20 Test Plates, Wash Buffer, Sample Diluent, Positive Control, Negative Control, Conjugate, TMB Substrate Solution, Stop Solution	270205
<i>cattletype</i> MAP Ab (5)	For 480 reactions: 5 Test Plates (strips), Wash Buffer, Sample Diluent, Positive Control, Negative Control, Conjugate, TMB Substrate Solution, Stop Solution	270803

Product	Contents	Cat. no.
<i>cattletype</i> MAP Ab (20)	For 1920 reactions: 20 Test Plates, Wash Buffer, Sample Diluent, Positive Control, Negative Control, Conjugate, TMB Substrate Solution, Stop Solution	270805
<i>cattletype</i> Milk Prep Kit (50)	Precipitation Reagent, Neutralization Buffer, Matrix, Elution Buffer, Spin Filters, Collection Tubes	271906

QIAGEN offers a range of ELISA kits and real-time PCR and real-time RT-PCR kits for the detection of animal pathogens. Visit **www.qiagen.com/Animal-and-Veterinary-Testing** for more information about *bactotype*[®], *cador*[®], *cattletype*, *flocktype*[®], *pigtype*[®] and *virotype*[®] products.

For up-to-date licensing information and product-specific disclaimers, see the respective QIAGEN kit handbook or user manual. QIAGEN kit handbooks and user manuals are available at **www.qiagen.com** or can be requested from QIAGEN Technical Services or your local distributor.

Notes

Notes

Limited License Agreement for *cattletype* BHV1 gB Ab

Use of this product signifies the agreement of any purchaser or user of the product to the following terms:

1. The product may be used solely in accordance with the protocols provided with the product and this handbook and for use with components contained in the kit only. QIAGEN grants no license under any of its intellectual property to use or incorporate the enclosed components of this kit with any components not included within this kit except as described in the protocols provided with the product, this handbook, and additional protocols available at www.qiagen.com. Some of these additional protocols have been provided by QIAGEN users for QIAGEN users. These protocols have not been thoroughly tested or optimized by QIAGEN. QIAGEN neither guarantees them nor warrants that they do not infringe the rights of third-parties.
2. Other than expressly stated licenses, QIAGEN makes no warranty that this kit and/or its use(s) do not infringe the rights of third-parties.
3. This kit and its components are licensed for one-time use and may not be reused, refurbished, or resold.
4. QIAGEN specifically disclaims any other licenses, expressed or implied other than those expressly stated.
5. The purchaser and user of the kit agree not to take or permit anyone else to take any steps that could lead to or facilitate any acts prohibited above. QIAGEN may enforce the prohibitions of this Limited License Agreement in any Court, and shall recover all its investigative and Court costs, including attorney fees, in any action to enforce this Limited License Agreement or any of its intellectual property rights relating to the kit and/or its components.

For updated license terms, see www.qiagen.com.

Trademarks: QIAGEN[®], Sample to Insight[®], *bactotype*[®], *cattletype*[®] *flocktype*[®], *pigtype*[®], *virotype*[®] (QIAGEN Group); Milli-Q[®] (Merck KGaA, Darmstadt, Germany). Registered names, trademarks, etc. used in this document, even when not specifically marked as such, are not to be considered unprotected by law.

HB-1588-005 © 2015 QIAGEN, all rights reserved.

Australia • techservice-au@qiagen.com
Austria • techservice-at@qiagen.com
Belgium • techservice-bnl@qiagen.com
Brazil • suportetecnico.brasil@qiagen.com
China • techservice-cn@qiagen.com
Denmark • techservice-nordic@qiagen.com
Finland • techservice-nordic@qiagen.com
France • techservice-fr@qiagen.com
Germany • techservice-de@qiagen.com
Hong Kong • techservice-hk@qiagen.com
India • techservice-india@qiagen.com
Ireland • techservice-uk@qiagen.com
Italy • techservice-it@qiagen.com
Japan • techservice-jp@qiagen.com
Korea (South) • techservice-kr@qiagen.com
Luxembourg • techservice-bnl@qiagen.com
Mexico • techservice-mx@qiagen.com
The Netherlands • techservice-bnl@qiagen.com
Norway • techservice-nordic@qiagen.com
Singapore • techservice-sg@qiagen.com
Sweden • techservice-nordic@qiagen.com
Switzerland • techservice-ch@qiagen.com
UK • techservice-uk@qiagen.com

www.qiagen.com